

Therapy Stool Care and Maintenance

Keep wheels free of dirt, hair and other loose fibres.

Do not clean any part of the Therapy stool with bleach or solvent cleaners.

Clean metal work with a slightly damp cloth.

Caution must be taken through use, transport and storage not to drop or knock the Therapy stool. Do not store other equipment on top of the stool or its accessories as this may cause damage.

Check the cover for signs of tearing or split seams. Wash at 40 degrees using standard non-bio detergents. Do not tumble dry. Allow plenty of time for the cover to dry out thoroughly.

Ensure the hand lever which controls the angle of the seat unit tightens up fully and holds the angle of the seat unit.

Check the plastic base for signs of cracks. If the base is cracked or heavily damaged, take the stool out of service and contact Quest 88.

Intended Use

Quest Therapy Stools are intended to be used as an sitting aid for physiotherapists.

DO NOT EXCEED WEIGHT LIMITS SET OUT ON THE FRONT OF THESE INSTRUCTIONS.

A LEVEL OF SUPERVISION APPROPRIATE TO THE USER'S ABILITY SHOULD BE APPLIED AT ALL TIMES

Do not use the therapy stool on any inclined surface or steps and stairs.

Do not store the therapy stool where it may block access to doors, walkways or fire exits.

The therapy stool has been tested and checked by Quest 88 limited engineers. Any adjustments or alterations which are not listed in these instructions should not be made without the authorisation of a Quest 88 limited representative. Any such adjustments will affect the stool's warranty. Quest 88 limited do not take any responsibility for any therapy stool which has been adapted or affected by actions not described in these instructions by any individual not authorised by Quest 88 limited.

Identification

Each therapy stool has a label with a QU number, date and the initials of the checker written on it. Do not remove this label. Always quote the QU number during any correspondence with Quest which refers to the therapy stool.

Quest 88 limited

Aston Street, SHIFNAL, Shropshire TF11 8DW
tel: +44 (0)1952 463050 fax: +44 (0)1952 463077

USER INSTRUCTIONS

Quest Therapy Stool

These are the user instructions for Quest Therapy Stools. Please read them carefully before assembling and using the Therapy Stool and store them in a safe place. Always refer to these instructions before adjusting the set up of the Therapy Stool, or if you feel the stool is not performing properly.

These instructions also include a guide to maintenance and general care. If followed, the Therapy Stool will provide trouble free service. However, in the event of any severe deterioration in performance, take the Therapy Stool out of service immediately and contact Quest 88 limited.

Quest Therapy Stools, Assembly


Quest Therapy Stools are delivered 'flat packed' and require a small amount of assembly. The stool consists of three parts; the seat unit, the height adjustment column and the base and castors.

These three items fit together without the need for tools.


Place the base, castors down, on the floor. Insert the black end of the height adjustment column into the circular hole in the center of the base. Push down on the column until it feels firm in the base.

Turn the seat unit upside down and locate the hole on the underside of the unit. Fit this hole onto the silver end of the height adjustment column and press down until the seat unit feels firmly in place.

Sit on the seat unit to fully locate the items together.

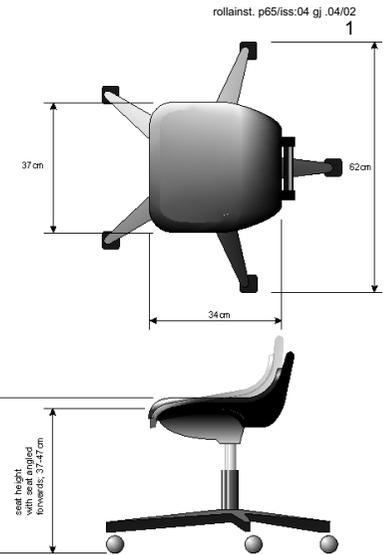

Maximum User Weight = 100kg

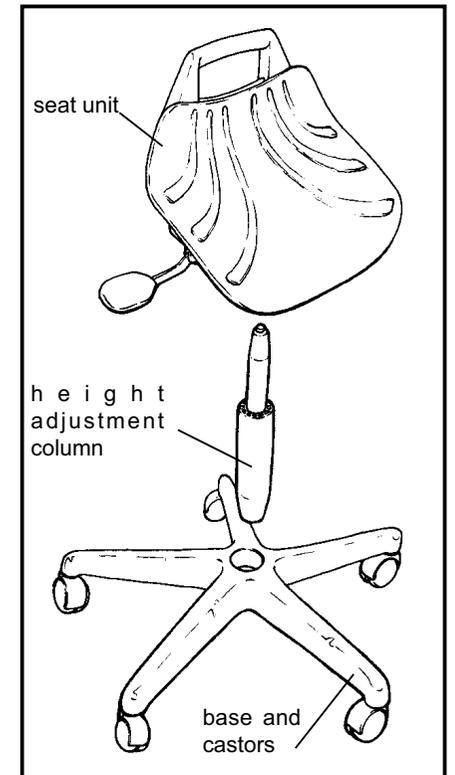

Setting Up for Use

Height Adjustment


The therapy stool has a 10cm range of height adjustment. The stool adjusts in the same way as most office chairs. The therapy stool can be adjusted down whilst the user is sat on the stool, however, the user will need to stand up to raise the height of the stool.

To adjust the height of the stool, lift up the lever on the right hand side of the stool. Adjust the stool to the desired height and let go of the lever. The height of the stool will lock once the lever is released.

Seat Angle Adjustment

The seat has a sprung loaded angle adjustment mechanism. When the seat is level, the spring is compressed, so once the lever which controls angle adjustment is released, the seat will automatically tilt forwards.

To release the angle adjustment mechanism, pull the lever up. When the mechanism is released, the seat can be tilted. Once the seat is at the desired angle, push the lever down to set the angle.

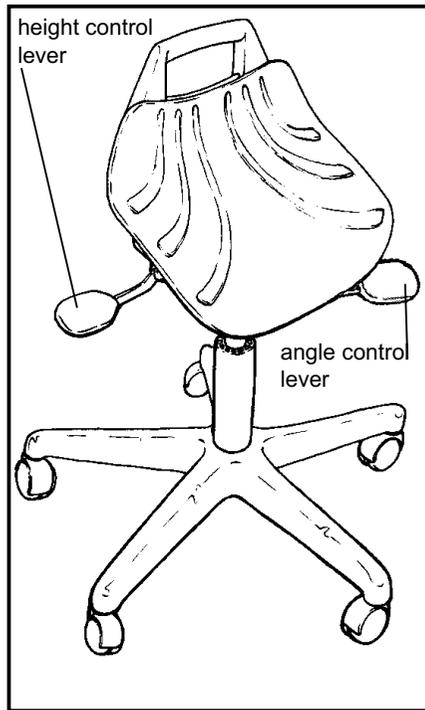

therapy stool

stool adjustments

Fitting the Loose Cover onto the Therapy Stool

The loose cover for the therapy stool is made from a waterproof, Teflon coated material which complies with all relevant fire safety standards. It is lined with a thin layer of foam to help cushion the seat.


The loose cover can be removed for cleaning, although regular cleaning with a damp cloth will keep the cover in good condition.

There is a pocket running around the underside of the loose cover. This pocket fits around the edge of the seat and hold sthe cover in place.


To fit the cover, fit the pocket over the front edge of the seat first. Keep pulling the pocket over one side of the seat, up to the handle on the back of the seat.

With the front edge and one side of the seat covered, carefully pull the cover over the other side, ensuring that the pocket is not creased or folded on the underside of the seat.

Hook the end of the pocket over the last exposed corner of the seat and smooth the loose cover down.

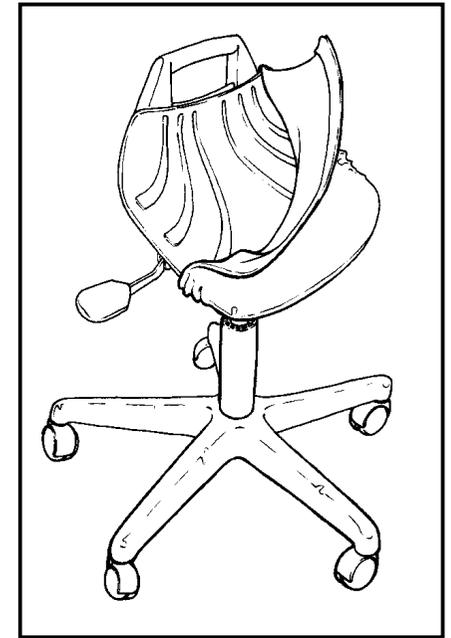

therapy stool loose cover fitting

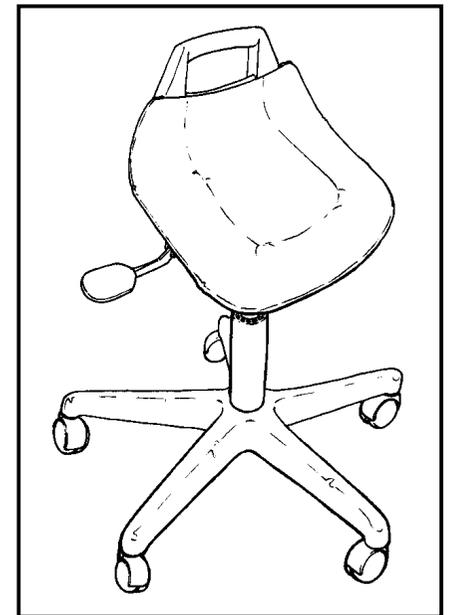

therapy stool with loose cover