

Quest 88 limited

Aston Street, SHIFNAL, Shropshire TF11 8DW
Tel.: +44 (0)1952 463050 Fax.: +44 (0)1952 463077

Quest 88 Limited



User Instructions for the Quest 88 folding pedal adaptor

These instructions are a supplement and should only be used in conjunction with the Draisin general user instructions and the Draisin Shopi supplementary instructions. Please read them carefully before using the assembly and store them in a safe place for future reference.

The folding pedal adaptor is an interface device that enables footplates to be folded away when used in-conjunction with folding pedals. Its most common application is on the Draisin Shopi tricycle which uses folding pedals to enable the tricycle to be folded for storage and transit.

Attaching Footplates to Folding Pedal



To remove footplate first release folding pedal lock by pinching/pulling to the outside of the pedal. This requires a reasonable amount of pressure.



Rotate the footplate away from the pedal body as the location pins clear the slots in the footplate mounting.



Remove footplate.

To refit the footplate to pedal reverse instructions apply.

If using a footplate adapted for ankle foot orthoses ensure the splint location guide is to the outside of the cycle so it does not interfere with the rotation of the pedal.