

Quest 88 limited

Aston Street, SHIFNAL, Shropshire TF11 8DW
Tel.: +44 (0)1952 463050 Fax.: +44 (0)1952 463077

These instructions are a supplement and should only be used in conjunction with the Quest 88 Ltd. Upi standing frame instructions. The instructions are for retro fitting new tri-glide straps to the Quest 88 Ltd. Upi standing frame.

The straps that you receive will be adjustable for fitting around the user and secured in place by folding back the Velcro strap on itself. If you are replacing the straps on just the hip support of your Upi stander, the strap kit will consist of two straps and two tri-glides as shown in fig.1. If you are replacing the hip and thoracic then the kit is double.

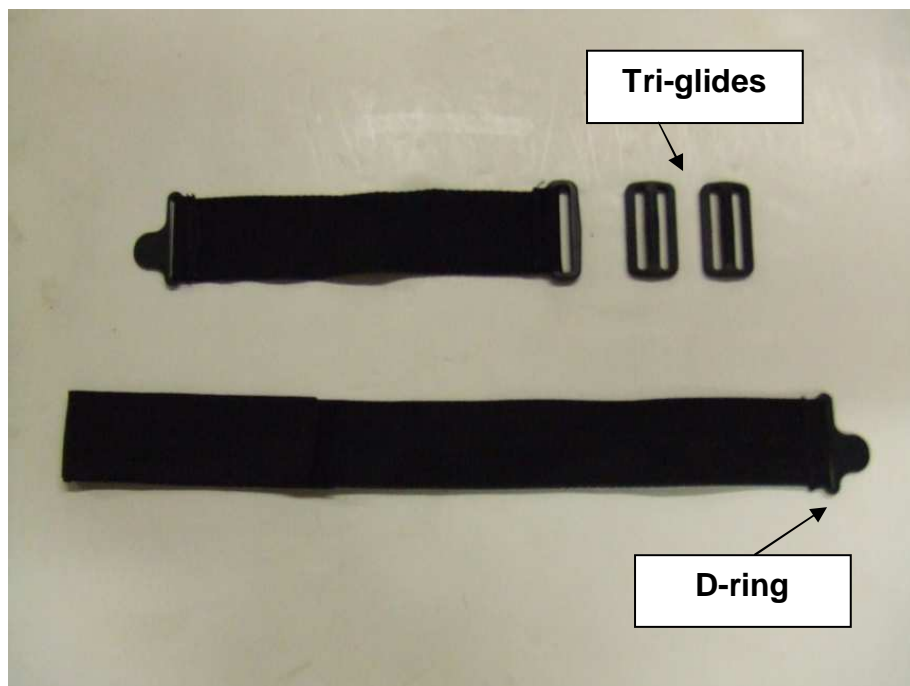


Fig. 1



Feed the d-ring through the slot in the lateral.



Feed the d-ring through the tri-glide.



Repeat for other lateral.



Please contact Quest 88 Ltd. should you need further clarification on any of the above points.