
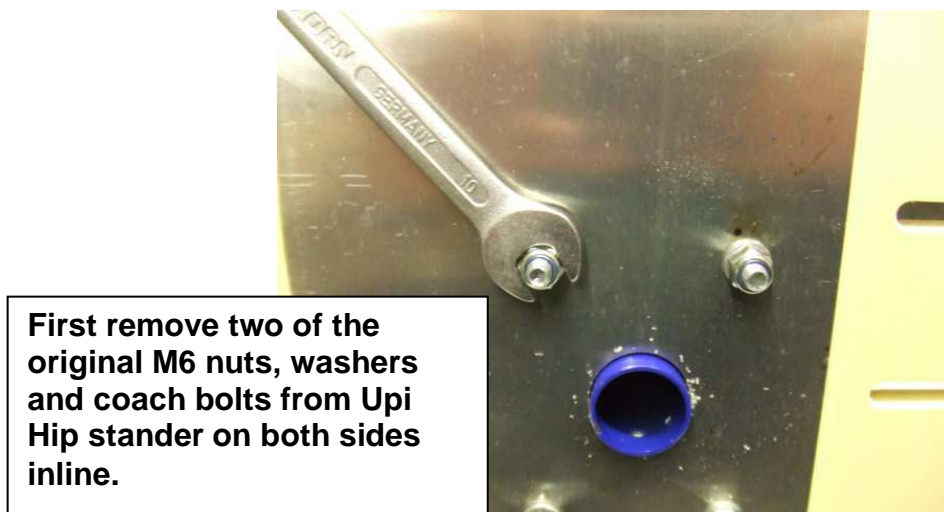


# Quest 88 limited

Aston Street, SHIFNAL, Shropshire TF11 8DW  
Tel.: +44 (0)1952 463050 Fax.: +44 (0)1952 463077


## **Stabilizer – Upi Hip Stander Supplementary Instructions.**

These instructions are a supplement and should only be used in conjunction with the Quest 88 Upi Hip stander user instructions. Please read them carefully before using the assembly and store them in a safe place for future reference. For details of use and adjustment of the Upi Hip stander, please refer to the main user instructions for the product.

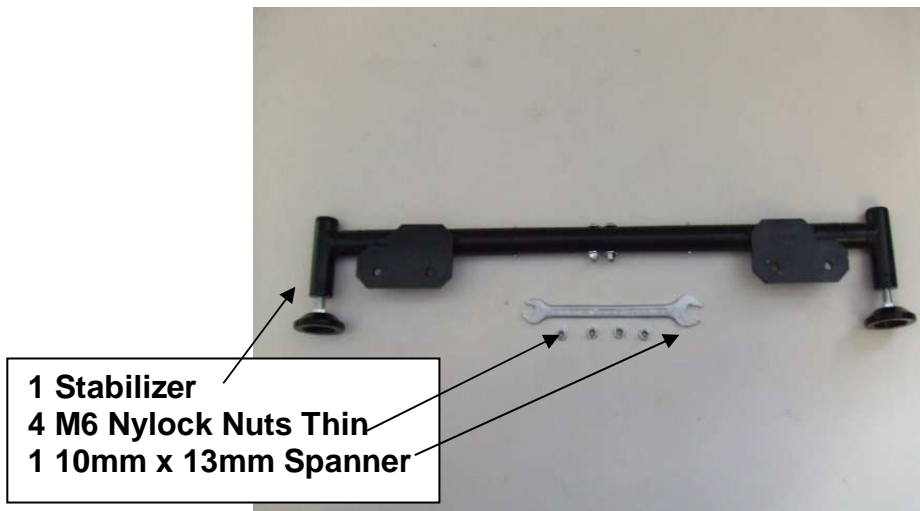



Fit stabilizer using existing coach bolts, washers and new M6 nylock nuts supplied.

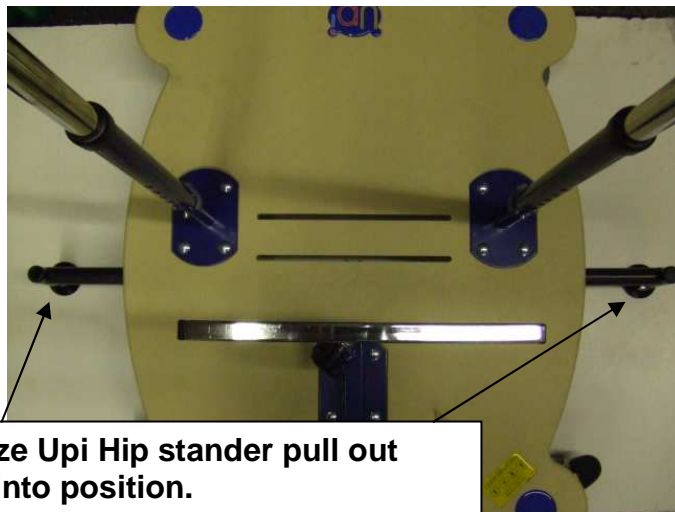

To stabilize Upi Hip stander pull out and lock into position.



Adjust feet to make contact with floor ensuring all 4 wheels of Upi Hip stander remain in contact with the floor.

Please contact Quest 88 should you need further clarification on any of the above points.